

# trophon – Protection by design

## Setting the standard for the broadest microbial efficacy

- ✓ The only automated ultrasound probe high-level disinfection (HLD) technology to meet mandatory microbial efficacy requirements for both FDA-clearance and CE-marking
- ✓ Meets criteria for bactericidal, mycobacterial, fungicidal and virucidal disinfection efficacy in accordance with AOAC International Official methods and ASTM International Standards<sup>1</sup>
- ✓ Tested against an extensive range of infectious pathogens, including those that cause STIs, such as chlamydia, gonorrhoea, herpes, HIV, hepatitis A, B and C, as well as HPV, *Clostridium difficile* spores and drug-resistant bacteria (MRSA and VRE)<sup>2,3</sup>



CE-marking: Manufacturers' declaration that the product meets EU standards for health, safety, and environmental protection. ASTM: American Society for Testing and Materials. AOAC: Association of Official Analytical Chemists. STIs: Sexually Transmitted Infections. HIV: Human Immunodeficiency Virus. HPV: Human Papillomavirus. MRSA: Methicillin-Resistant *Staphylococcus Aureus*. VRE: Vancomycin-Resistant *Enterococci*.

## Ultrasound manufacturers' reprocessing solution of choice

- The trophon Probe Compatibility program is conducted in partnership with ultrasound probe manufacturers<sup>4</sup>
- All probes listed as compatible have been approved by the manufacturer for use with trophon
- Over 1,000 probes compatible across 24 manufacturers
  - New features on trophon2 devices:
    - Universal cable clamp compatible with a wider range of cable diameters<sup>5</sup>
    - Integrated Probe Positioner (IPP).



<sup>4</sup>Compared to trophon EPR.

# trophon2 – The Global Standard of Care in ultrasound probe reprocessing



## DEMONSTRATED COMPLIANCE ACROSS THE WORKFLOW

- Automated and consistent HLD
- AcuTrace RFID technology

## EFFICIENT WORKFLOW INTEGRATION

- Point-of-care HLD
- Minimal “hands-on” reprocessing time
- Environmentally friendly by-products

## CUSTOMISED CONFIGURATION

- Adjustable wake-up/sleep timers
- Purge now or defer feature

## ENHANCED USABILITY

- Touch screen interface
- Setup wizard and settings menu

## BROADEST MICROBIAL EFFICACY

- Meets mandatory requirements
- Extensively tested against clinically-relevant species

## EXTENSIVE PROBE COMPATIBILITY PROGRAM

- Compatible with over 1,000 probes
- Approved by probe manufacturers

Everyday, approximately 80,000 patients worldwide are protected from the risk of cross-contamination because the ultrasound probe has been high-level disinfected with trophon. This equates to around 20 million patients annually!<sup>1</sup>

<sup>1</sup>As at December 2020.

**Nanosonics Limited**  
14 Mars Road, Lane Cove NSW 2066, Australia  
T: +61 2 8063 1600 E: info@nanosonics.com.au W: www.nanosonics.com.au

**Nanosonics, Inc. (Distributor USA and Canada)**  
7205 E 87th Street, Indianapolis, IN 46256, USA  
T: 1-844-876-7466 E: info@trophon.com W: www.nanosonics.us



References: 1. Vickery, K et al. J Infect Public Health 2014;7(2):153-160. 2. Ryndock E, et al. J Med Virol. 2016;88(6): 1076-1080. 3. trophon microefficacy brochure/internal test reports. 4. Nanosonics Probe Compatibility Program data 2019. 5. American National Standards Institute (ANSI), Association for the Advancement of Medical Instrumentation (AAMI). ST58: 2013 Chemical sterilization and high-level disinfection in health care facilities. 2013. 6. Canadian Standards Association (CSA) 2018. CAN/CSA-Z314-18. Canadian Medical device reprocessing. © 2020 Nanosonics Limited. All rights reserved. NAN0371, MM01628-US-&-CA-BR V1, April 2021.



## Delivering consistent protection across every high-level disinfection (HLD) cycle



# trophon® – Protection by design



- Nanosonics' trophon technology helps protect patients by delivering **automated and consistent high-level disinfection (HLD) of ultrasound probes with every cycle**
- For endocavitary and surface probes used in semi-critical and critical procedures, the fully-enclosed trophon technology#:
  - Uses high-frequency ultrasonic vibrations to generate a sonically-activated, hydrogen peroxide (H<sub>2</sub>O<sub>2</sub>) mist that kills bacteria, fungi, viruses and mycobacteria
  - Accesses all surfaces of the probe, including the body and handle, ensuring all crevices and imperfections are high-level disinfected
  - Produces only water and oxygen as the environmentally-friendly by-products of the HLD cycle.

**trophon technology delivers protection for patients, staff and the environment**

**Due to greater reliability and reproducibility, most global guidelines now recommend automated methods over manual disinfection**

# trophon2 – Streamlines workflow efficiencies

## trophon2 devices allow an integrated point-of-care workflow



- As a fully-enclosed compact system, trophon2 devices make reprocessing in the patient environment possible.
- Can be placed in patient examination rooms with no special ventilation requirements
  - Reduces the risk of exposure to hazardous chemicals, fumes and spills often associated with wipes and soaks
  - Only gloves required for PPE
  - Minimal hands-on time required.

**Frees up your time for enhanced patient workflow** ✓

## trophon2 devices are configurable and easy to use

New features and settings with trophon2 enhance the workflow even further:


- **Daily warm-up:** Adjustable “wake up” times so it’s ready to start when your day begins
- **Sleep timer:** Enters power saving mode when not in use. Adjust in hourly increments or option to turn off
- **Purge now or defer:** Purge can be deferred to later in the day when disinfectant has reached 30-day expiry in the device
- **Large touch screen interface:** First time setup wizard and settings menu access.

# trophon2 – Delivers demonstrated compliance



## trophon2 devices automatically demonstrate compliance with AcuTrace® RFID technology

- Accrediting agencies look for traceability processes that are compliant with evidence-based guidelines
- trophon2 with AcuTrace RFID technology delivers digital traceability across the reprocessing workflow:
  - Digital audit-ready records reduces the need for manual records, saving time and preventing human error
  - Records probe, operator, consumables and cycle data
  - Demonstrates user-compliance in accordance with AAMI and CSA standards.<sup>5-6</sup>

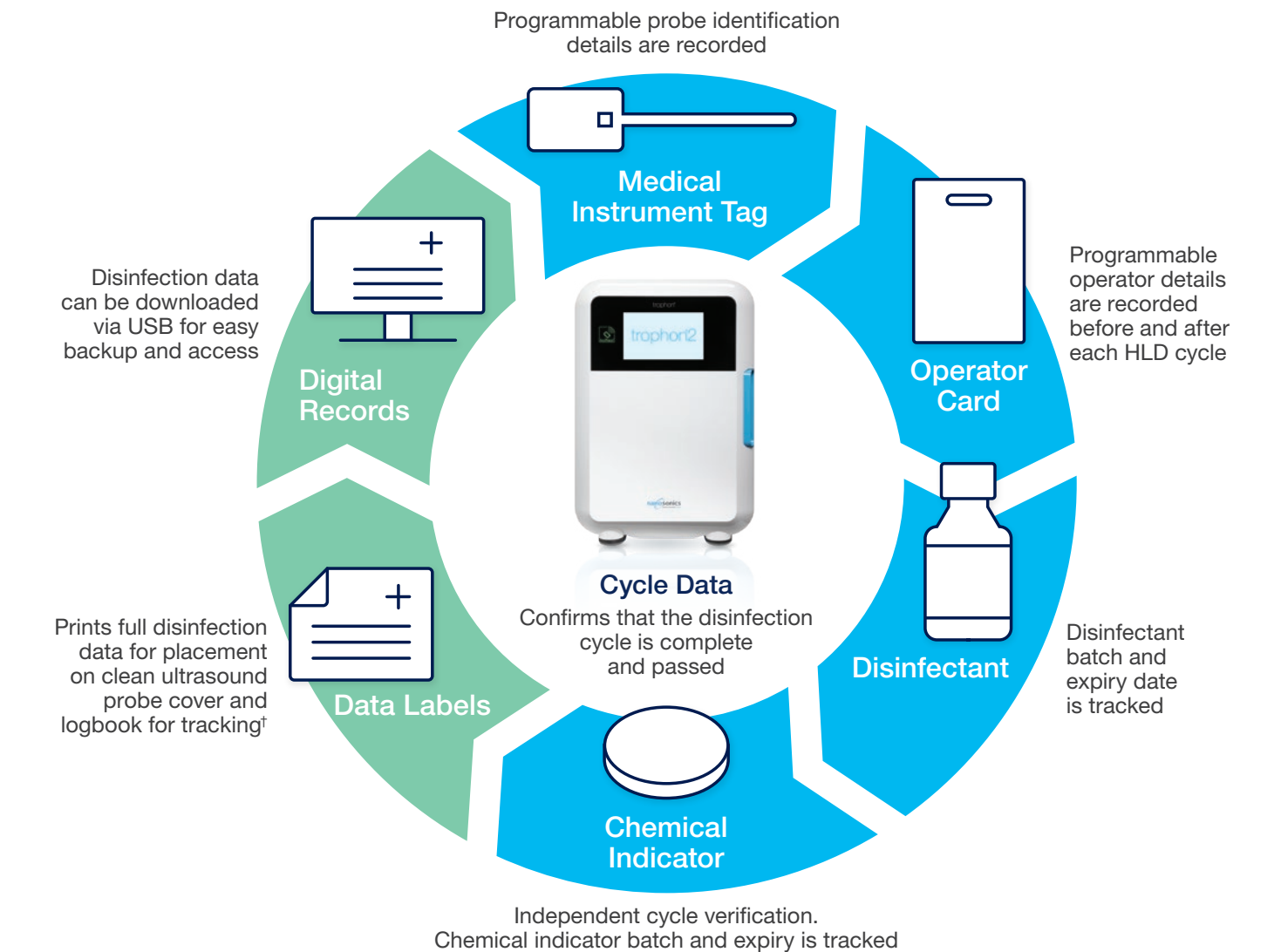

\*The trophon family includes the trophon EPR and trophon2 devices which share the same core technology of sonically-activated hydrogen peroxide.

\*When used in conjunction with the trophon printer.